



Uriscreen™

RAPID SCREEN FOR URINARY TRACT INFECTION (UTI)

Test for bacteriuria and presence of somatic cells in urine

- ▶ Visual results within 2 minutes
- ▶ Easy to use; requires no instrumentation
- ▶ Perfect UTI screen for outpatient settings
- ▶ Improved performance over routine urinalysis
- ▶ Exceptionally sensitive for low bacterial counts (5×10^4 CFU/ml at 93% sensitivity)
- ▶ Efficient way to screen out negative specimens and determine need for further testing
- ▶ Early diagnosis prevents unnecessary treatment and minimizes costly complications of UTI

**CLIA
Waived**





Uriscreen™

- ▶ Accutest® Uriscreen™ is a rapid screening test which combines the detection of both bacteriuria and pyuria (the presence of somatic cells in urine), by detecting catalase activity (an enzyme present in many eukaryotic cells).
- ▶ Accutest® Uriscreen™ provides physicians' offices, clinical laboratories and outpatient sites with the ability to provide fast and accurate screening for urinary tract infections.
- ▶ Accutest® Uriscreen™'s negative predictive value (NPV) of at least 95%, was higher than the NPV of the urine dipstick method (see references).
- ▶ Accutest® Uriscreen™ is a cost-effective screening procedure, which can be used to complement conventional culture methods.
- ▶ CPT Code: 81007QW

Procedure:



Urine specimen is added to a test tube ▼



4 drops of hydrogen peroxide solution are added to the tube and mixed ▼



The appearance of white foam indicates a positive result

Ordering information

Cat. Number	Product Name	Number of Tests
ID601	Accutest Uriscreen™	20

References

1. Carroll K. C. et. al., 1994, "Laboratory Evaluation of Urinary Tract Infection in an Ambulatory Clinic", Am. J. Clin. Pathol.; 101:100-103.
2. Hagay, Z. et. al., "Uriscreen, a Rapid Enzymatic Urine Screening Test: Useful Predictor of Significant Bacteriuria in Pregnancy".

Negative



Positive



Indicates presence of UTI (bacteriuria/hematuria/pyuria)

JANT PHARMACAL CORPORATION

16255 Ventura Blvd., #505, Encino, CA 91436
800.676.5565 818.986.8530 Fax 818.986.0235
www.accutest.net info@accutest.net

Accutest is a registered trademark of Jant Pharmacal Corporation