



INFLUENZA A+B TESTS

ACCU... TEST before you treat.

Immunochromatographic, Rapid Tests for the Qualitative Detection of Influenza A&B Viruses in Nasopharyngeal Secretion

- Single Tests**
- Influenza Type A Virus (H1N1, H3N2, H5N1 etc.)
 - Influenza Type B Virus

- Dual Tests**
- Test for both Influenza Type A and B Viruses

- Performance**
- Rapid results in minutes, for better clinical management
 - Same day detection & treatment decisions
 - Point of care testing

- Convenient**
- Simple visual membrane test
 - Easy sample collection and assay procedure
 - Room temperature storage
 - Kit includes test device, filter tubes, dropper bottles, swabs and instructions





INFLUENZA A+B TESTS

Physician's Procedure

Assay Procedure

1. Bring all reagents to room temperature.
2. Remove the test device from the pouch by tearing along the split.
3. Using the dropper filter-tube, add 6-7 drops (200uL) of the extracted solution into the sample well.
4. Read the results of the test 15 minutes after addition of the sample to the device.

Reading Test results

A. NEGATIVE



Only one colored band appears in the control zone.
No band appears in the test zone.

B. POSITIVE

In addition to the control band, a clearly distinguishable band also appears in the test zone.

C. INCONCLUSIVE

If there is no distinct color band visible in the control zone, the test is inconclusive. The test should be repeated.

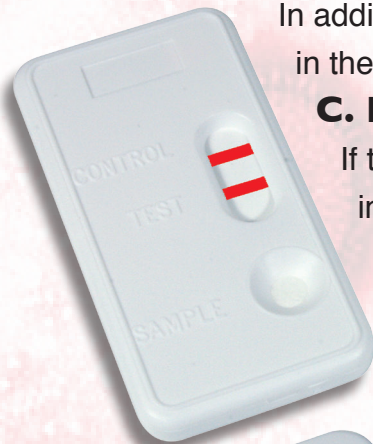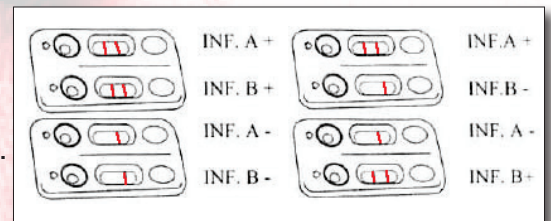

Single Tests

- Accutest® Influenza A - 20 Test Kit Catalog: #IE43029
- Accutest® Influenza B - 20 Test Kit Catalog: #IE43129

Dual Tests

- Accutest® Influenza A&B - 20 Test Kit Catalog: #IE43229

Kit Includes

- 20 Test Devices
- 20 Swabs
- 20 Filter Tubes with Cap
- 2 Dropper Bottles with 11mL Extraction Solution
- 1 Instruction Leaflet

DISTRIBUTED BY:



JANT PHARMACAL CORPORATION

16255 Ventura Blvd., #505, Encino, CA 91436
800.676.5565 818.986.8530 Fax 818.986.0235
www.accutest.net info@accutest.net

Accutest is a registered trademark of Jant Pharmacal Corporation