



Single or Dual iFOBT Tests

... Accutest Offers a Choice

An important aid in the diagnosis of colorectal cancer.

- Innovative immunological Fecal Occult Blood Test that **allows two samples to be tested - economically - on the same test card.**
- Testing more than one sample from different bowel movements improves accuracy and is recommended by the American Cancer Association, since polyps bleed intermittently.

Economical:

- Two samples tested for less than many single tests

Performance:

- Immunoassay test specific to human hemoglobin
- Sensitivity 99.2%, Specificity 96.7%, Accuracy 98.3%
- Cutoff is 50ng hHb/ml
- Extended stability of sample in buffer solution (18 days room temperature/6 months refrigerated)

Compliance:

- No dietary or drug restrictions
- Easy to use collection materials

Easy to use:

- Rapid results, CLIA - waived, easy to read

Reimbursement:

- Significantly greater with immunoassay test
- 2010 Medicare Reimbursement \$22.78 (vs. \$4.66 with Guaiac test)


CPT Codes: 82274QW or G0328QW




accutest® iFOBT Test

Physician's Procedure for Dual-Sample iFOBT Test


1. Shake the sample collection tube vigorously to ensure a good liquid suspension

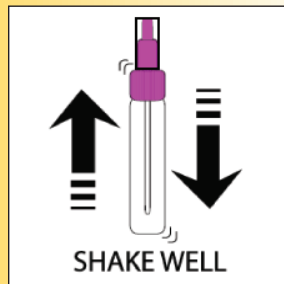


2. Holding the tube upright, unscrew the clear tip cover.

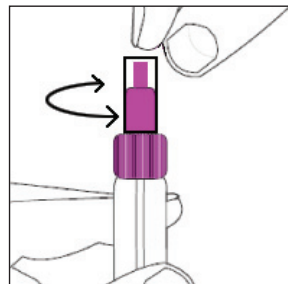


3. Dispense four (4) drops of suspension from each tube into appropriate sample well:

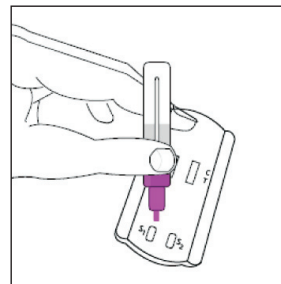

4. Read results within 5-10 minutes after adding the extraction solution

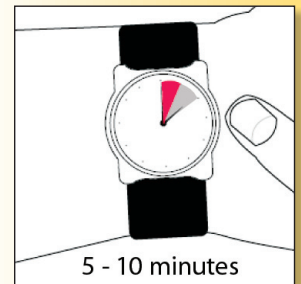

A. Four (4) drops from sample tube one (1) should be placed into sample well one (S1)
 B. Four (4) drops from sample tube two (2) should be placed into sample well two (S2)

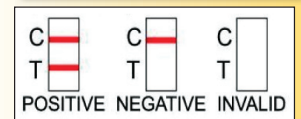

Dual Sample Test

Accutest iFOBT - 40 Test Kit (20 cassettes) with 25 patient specimen collection kits
 Catalog No. CS612

Accutest iFOBT - 25 patient specimen collection kits
 Catalog No. CS632

Single Sample Test

Accutest iFOBT - 20 Test Kit with 25 patient specimen collection kits
 Catalog No. CS601

Accutest iFOBT - 25 patient specimen collection kits
 Catalog No. CS627



Accutest iFOBT - 25 test devices and 25 sample collection tubes
 Catalog No. CS625

DISTRIBUTED BY:



JANT PHARMACAL CORPORATION

16255 Ventura Blvd., #505, Encino, CA 91436
 800.676.5565 818.986.8530 Fax 818.986.0235
 www.accutest.net info@accutest.net

Accutest is a registered trademark of Jant Pharmacal Corporation